



Pacer

Gait training has come a long way since we introduced our first model almost 30 years ago, and our Pacer continues to evolve and improve to meet our customers' needs. Over the years your suggestions have helped us keep our design at the cutting edge.

The Pacer, because it can be configured in so many ways, can accommodate clients of every ability, in any environment. It's the versatility and dependability you've come to expect from Rifton.



Find letters of medical necessity and informative articles at: www.rifton.com/pacer

students have a wide range of motor issues. Many need a gait trainer that allows them more freedom of movement. Sometimes with our old equipment, it seems like they're fighting against the device, but Rifton's dynamic Pacer allows and even shapes the natural movement that occurs with walking. It's fabulous.

Nikki Cornell, MPT Clovis Unified School District, CA

The Pacer (dynamic shown here) is easily configured to meet your clients' every need. Choose between dynamic and standard upper frames and pair them with standard, utility and treadmill/stability bases.




Pacer features









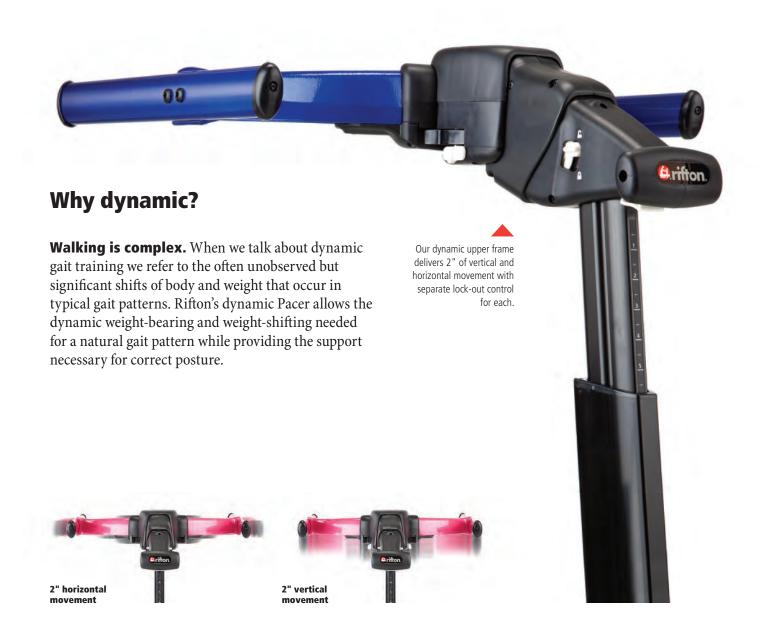

Perfect positioning made easy

The single-column upper frame

features a locking gas spring that adjusts to any height on the indexed column.





Combining the fluid height adjustments on the MPS and upper frame brings the seat low enough for transfers from the seated position.



Once the transfer is completed, caregivers raise the upper frame to bring the client into an upright position.



The Multi-Position Saddle (MPS)

is the ultimate pelvic positioning accessory. Adjustable in five directions to accommodate each client's posture, the MPS easily attaches to both the dynamic and standard upper frames.





Hip corral depth and height adjustments move
the hip corral to provide
cushioning and support for
the pelvis at the level of ASIS.

Adjustment range: Large: depth 3", height 3" Med/Small: depth 3", height 2"



Saddle depth adjusts to optimally position the pelvis in relation to the upper body to encourage weight-bearing and forward motion during ambulation.

Adjustment range: Large: 5", Med/Small: 4"



Saddle height adjusts to fine-tune the amount of weight-bearing support during gait.

Adjustment range:

Large: 8½", Med: 8½", Small: 5½"

Saddle angle is adjustable to position the pelvis in anterior or posterior tilt for the most effective gait pattern.

Adjustment range:

Large: 15° forward and 15° back Med/Small: 7° forward and 7° back





Using the height adjustment on the MPS, the caregivers raise the client's pelvis to the optimal height for weight-bearing and step-taking.



With the tilt adjustment on the MPS, the caregivers position the client's pelvis for the most effective gait pattern.

Infinite control for effective movement

The Pacer casters' five functions let you fine tune the Pacer's movement as your clients gain control.

Standard base casters



Utility base front caster

Swivel lock

Utility base rear wheel





A measurable improvement

The odometer, available as an option on every Pacer base, records a client's progress in either feet or meters. No more counting tiles!



Front caster with odometer



Transportable

The detachable upper frame makes it easy to stow and transport the Pacer. Separable components allow you to bring the right base for each client and every setting.



NEW

Go minimal

The new Pacer chest pad and handlebars form a lightweight configuration perfect for lower GMFCS levels. Available for small, medium and large Pacer sizes, these components allow easy transfers and quick adjustment.







Posterior position



Posterior positioning is a natural progression towards independent movement, and it gives the users improved access to their environment.

Note: The standard base is recommended for posterior positioning. When the utility base is reversed, steering is more difficult and the directional lock feature will not work.

Reverse the MPS

The MPS has been designed to allow reverse positioning of the saddle, which enables gait practice with the frame in the posterior position. For instructions on how to place the MPS in the posterior position, visit www.rifton.com/MPSposterior.



Utility base

Easy rolling over thresholds or thick carpet (or even outdoors on gravel or grass) is now yours with the large-wheeled utility base, letting you go places you never could before.





Treadmill/stability base



Now you can use your Pacer on the treadmill. Available in two widths, these bases straddle almost any treadmill and can be used with the large, medium and small upper frames. It's not just for treadmill use; overground this base provides all the functions of our standard base but with increased width for better stability.



The treadmill/stability base has 7½" of height adjustment to compensate for the height of a treadmill.



Components



Dynamic upper frame provides dynamic weight-shifting and dynamic body-weight support, encouraging natural gait patterns.



Standard upper frame is a great choice for clients who do not need dynamic movement. It accommodates all the Rifton prompts you're familiar with.



Standard base

Designed to be used on smooth surfaces.

Caster size: 5½"



Utility base

For indoors or out, this base is the best choice for navigating thresholds, lawns, gravel paths or chipped playgrounds. No downhill though!

Caster/wheel size: front casters 8", rear wheels 11½"



Treadmill/stability base

Designed to accommodate most treadmills, this base lets a client practice stepping on a treadmill before progressing to overground ambulation. It's also a good choice for clients who need a wider, more stable base for gait training.

Caster size: 5½"





Arm prompts

Versatile arm prompts include height, rotation, angle, lateral proximity and forward/backward adjustments to accommodate numerous positioning requirements.

Padded surface area Adjustment range

small 41/2"x 8" large 5"x 9"

(elbow to fist): small 9"-121/2"

large 101/2"-15"





Arm platforms

Arm platforms offer a simple alternative for clients who need less arm positioning. Platforms attach to our adjustable arm prompt base and are made of closed-cell molded foam for padded comfort and easy cleaning.

Padded surface dimensions: 131/2" x 51/2"





Hand brakes

The hand brakes combine both running and parking brake functions in a simple leveraction control. The brakes attach to the arm prompts and can be quickly removed when not needed.

Note: Since retrofitting is complex, we recommend ordering brakes with the original purchase.





Hand loops

For less involved clients, hand loops are a simpler option for upper extremity support.





Chest prompt

The padded chest prompt holds the client's torso securely at the angle of your choice. It is available in three sizes.

Range of circumference	Pad heigh
small 14"-28"	small 4½"
med 22"-40"	med 61/2"
large 28"-50"	large 8"



Components (continued)



Chest pad

The chest pad is ideal for less involved users. With its single support strap and double angle adjustment it allows easy transfers and quick positioning.

Range of circumference	Pad height
small 14"-28"	small 4"
med 22"-40"	med 5"
large 28"-50"	large 6"





Handlebars

For users who need less arm support, the new handlebars attach quickly and adjust easily. They pair perfectly with the simple chest pad.





Multi-position saddle

The multi-position saddle (MPS) enables precise positioning of the pelvis. Adjustable in five directions to accommodate each client's posture, the MPS attaches to both the dynamic and standard upper frames.

Dimensions: see page 9





Pelvic support

For clients who need less abduction, the pelvic support is a softer alternative to the hip positioner. It provides weight—bearing assistance and freedom of movement while clients build strength and stamina.

Dimensions: see page 20





Hip positioner

The hip positioner enables natural pelvic movement and lower extremity weight-shifting, and provides good abduction. A pad is available for added comfort.

Dimensions: see page 20







Thigh prompts


Thigh prompts swing with the user's stride and prevent leg scissoring. They are adjustable in the medial and lateral directions and prevent the user from turning within the Pacer frame.

Length (from clamp to strasmall 5"-8" large 5"-11"

Length (from clamp to strap) Max. leg circumference

small 15" large 20"



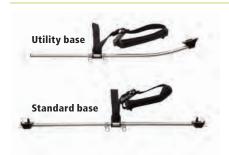

Ankle prompts

Ankle prompts have comfortably padded ankle straps and spring adjusters to control stride length and placement. Adjust the strap length to determine foot separation and prevent scissoring.





Handholds

The ergonomic handholds are heightadjustable, and can be placed anywhere along the top bar of the frame. Handholds can be used by caregiver or client.





Communication tray

Our redesigned communication tray features a new mounting system with an improved range of positions. It is great for holding tablets and other mobile devices.

Inside dimensions: 8½"x 11" Max. working load: 10 lb





Attendant guide bar

The guide bar is made of lightweight, high strength aluminum alloy. It attaches to the front tube of the standard or utility base with a quick-release clamp, and enables the caregiver to guide the Pacer from in front or behind.





Accessories tote

A tough and attractive tote bag big enough (14"x 16"x 9") to keep all your Pacer components together when not in use. Made of black ballistic nylon with a zippered top, here's a bag that will hold up to whatever you throw in it and still look great.




Pacer dimensions


Key user dime	ensions (inches)	K610 mini	K620 small	K630 medium	K640 large	K650 XL
Elbow height		151/2-201/2	181/2-271/2	24-35	32-47	34–49
M	ey user dimension: elbow height easure the vertical distance from the bent e hile the user is standing upright.	bow to the floor				
	noose the size that allows for growth.					
Item dimension	s (inches)	K610 mini	K620 small	K630 medium	K640 large	K650 XL
Floor to top of arm prompt		151/2-201/2	181/2-271/2	24-35	32-47	34-49
Floor to top of MPS	(not tilted)*	N/A	10-20	91/2 - 231/2	17 – 35	191/2 – 371/2
Overall width	Standard base:	201/2	23	26	28	311/2
	Utility base:	N/A	N/A	30	32	N/A
	Narrow treadmill/stability base:	N/A	35	35	35	N/A
	Wide treadmill/stability base:	N/A	40	40	40	N/A
Overall length	Standard base:	221/2	26	30	36	39¾
	Utility base:	N/A	N/A	36	401/2	N/A
	Treadmill/stability bases:	N/A	41	41	41	N/A
Frame height	Standard & utility bases:	12½ 16-21 21½-27 28½-38½ 31-		31-41		
	Treadmill/stability bases:	N/A	23-35	231/2-361/2	30-47	N/A
Base height	Standard & utility bases:	N/A	11	141/2	16	181/2
(without upper fran	ne) Treadmill/stability bases:	N/A	21	21	21	N/A
Frame weight (lbs)	Standard base:	101/4	111/2	15	151/2	221/2
	Utility base:	N/A	N/A	181/2	191/4	N/A
	Narrow treadmill/stability base:	N/A	271/2	271/2	271/2	N/A
	Wide treadmill/stability base:	N/A	28	28	28	N/A
	Standard upper:	N/A	6	71/2	91/2	11
	Dynamic upper:	N/A	71/2	111/2	131/2	15
Dynamic upper mo	vement Vertical:	N/A	11/4	2	2	2
	Horizontal:	N/A	1½	2	2	2
Maximum treadmil	width Narrow treadmill/stability base:	N/A	29	29	29	N/A
	Wide treadmill/stability base:	N/A	34	34	34	N/A
Maximum treadmil	height Treadmill/stability bases:	N/A	11	11	11	N/A
Maximum working	load (lbs)	50	75	150	200	250

^{*}This measurement is taken at the lowest frame and MPS height, and the highest frame and MPS height.

Pelvic support

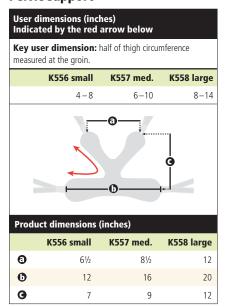

Hip positioner

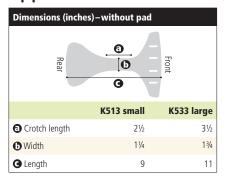



The Pacer comes in your choice of five colors.



2020 order form

Use dimension chart to select appropriate size.

	ise dimension chart to select appropriate size.		K610 mini K620 small K		K630 medium		K640 large		K650 XL				
* C	* Color		Red		Red		Red		Red		Red		
				Blue		Blue		Blue		Blue		Blue	
			СНООЅЕ	Gray		Gray		Gray		Gray		Gray	
		æ	Lime		Lime		Lime		Lime		Lime		
				Pink		Pink		Pink		Pink		Pink	
* U	pper frame	£	Dynamic	N/A		K621	\$1,050	K631	\$1,450	K641	\$1,630	K651	\$2,100 🔲
		СНООЅЕ	Standard	N/A		K622	\$620 🗌	K632	\$850 🗌	K642	\$1,030 🗌	K652	\$1,500 🔲
		오	With odometer	K613	\$890 🗌	K623	\$555 🗌	K633	\$695 🗌	K643	\$870 🔲	K653	\$1,275
		CHOOSE	Without odometer	K614	\$745 🗌	K624	\$390 🔲	K634	\$525 🔲	K644	\$705 🗌	K654	\$1,110
* Ø	Utility base		With odometer	N/A		N/A		K635	\$960 🗌	K645	\$1,130	N/A	
Base	(larger wheels)	СНООЅЕ	Without odometer	N/A	***************************************	N/A		K636	\$790 🔲	K646	\$965 🗆	N/A	
	Treadmill/stability base		Standard	N/A		K676	\$990 🗆	K676	\$990 🗆	K676	\$990 🗆	N/A	
	(comes in gray only)	CHOOSE	Wide	N/A	*	K678	\$1,110	K678	\$1,110	K678	\$1,110	N/A	
Arn	n supports (pair)		Small arm prompts	K512	\$485 🗌	K512	\$485	K512	\$485	K512	\$485	K512	\$485 🔲
7.	• 4· · · · · · · · · · · · · · · · · · ·	£	Large arm prompts	N/A	⊔	N/A	⊔	K532	\$520 🗆	K532	\$520 🗆	K532	\$520 🗆
		снооѕе	Arm platforms	N/A		N/A		K697	\$350	K697	\$350 🗆	K697	\$350
			Arm platforms w/ handgrips	N/A	_	N/A		K696	\$400 🗆	K696	\$400 🗆	K696	\$400 🗆
Har	ndbrake for utility base (requires a	arm sun] 1 31	N/A		N/A		K687	\$285	K688	\$310 🗆	N/A	Ţ.00 <u></u>
	w! Handlebars (pair)		Small	K625	\$185 🗌	K625	\$185	K625	\$185	K625	\$185	K625	\$185 🗌
	(pair)	СНООЅЕ	Large	N/A	*103 <u></u>	N/A	\$100 🗀	K675	\$195	K675	\$195	K675	\$195 🗆
Har	nd loops (pair)	m	Luige	K579	\$190 🗆	K579	\$190 🗆	K579	\$190	K579	\$190	K579	\$190 🗆
			Small	K514	\$510 🗆	K573	\$510	K514	\$510 🗆	N/A	1000	N/A	100
Chest prompt		Medium	N/A		K514	\$550 🗆	K514	\$550 🗆	K524	\$550 🔲	K524	\$550 🔲	
		3SO		N/A	_	N/A	٦٥٥٥ ـــ	K534	\$595	K524	\$595	K534	\$595 🗆
No	New! Chest pad		N/A		K629	\$280 🗆	K639	\$300 🗆	K649	\$320 🗆	N/A	\$232 🗔	
	Iti-position saddle		Small	N/A		K672	\$850	N/A	3300 🗀	N/A	\$320 🗀	N/A	
IVIU	iti-position saudie	CHOOSE	Medium	N/A		N/A	1000 🗀	K673	\$875 🗌	N/A		N/A	
		Large	N/A	_	N/A		K674	\$910 🗆	K674	\$910 🔲	K674	\$910 🗌	
Dol	vic support (includes handholds)		Small	K556	\$395 🗆	K556	\$395 🗆	K556	\$395	N/A	3910	N/A	ا ۱۱۹۴
rei	vic support (includes nandholds)	CHOOSE	Medium	ļ	1232 🗀					ļ	¢11E 🗆	ļ	
		OSE		N/A N/A		K557 N/A	\$415	K557 K558	\$415 _ \$430 _	K557	\$415 \ \$430 \	N/A K558	\$430 🗌
µ;	nositioner (includes handholds)	요	Large	K513	\$395 🗌	K513	\$395 🗌	K513	\$395	K513	\$395	K513	\$395
Hip positioner (includes handholds)		HOOSE		ļ	*323 🗀	N/A	1020			ļ			\$395 🔲
Hip positioner pad			Large	N/A	\$100 □		\$100 D	K533	\$430	K533	\$430 🗆	K533	
пір	positioner pau	CHOOSE	Small	K541	\$100 🗌	K541	\$100 🗌	K541	\$100 🗌	K541	\$100 🗌	K541	\$100 🗌
Th:	ah prompte (pair)		Large	N/A		N/A V515	\$200 🗆	K542	\$135	K542	\$135	K542	\$135 🗌
Thigh prompts (pair)		СНООЅЕ	Small	N/A		K515	\$380 🗌	K515	\$380 🗌	K515	\$380 🗌	N/A vs25	¢4E∩ □
ا م	VIO prompts for standard base /s		Large	N/A	\$160 D	N/A	\$170 I	K535	\$450	K535	\$450 🗆	K535	\$450 🗆
Ankle prompts for standard base (pair) Ankle prompts for utility base (pair)		K681	\$160 🗌	K682	\$170 🗌	K683	\$185	K684	\$200	K684	\$200 🗆		
, , ,			N/A		N/A	\$205 🗆	K693	\$185	K694	\$205	N/A		
Ankle prompts for treadmill/stability base (pair)			N/A	¢140 🗀	K664		K664	\$205 🗆	K664	\$205	N/A	¢110 □	
Additional handholds (pair)			K519	\$110 🗆	K519	\$110 🗆	K519	\$110 🗆	K519	\$110 🗆	K519	\$110 🗆	
Communication tray			K518	\$370	K698	\$370	K698	\$370	K698	\$370	K698	\$370	
Attendant guide bar		K589	\$140 🗆	K679	\$140 🗆	K689	\$140	K689	\$140	N/A	610-		
Rifton accessories tote		K522	\$120 🗌	K522	\$120 🗌	K522	\$120 🗌	K522	\$120 🗌	K522	\$120 🗌		
Tot	tal			K610	\$	K620	\$	K630	\$	K640	\$	K650	\$

^{*} Required