



Tricycles

Want to see your clients smile? Give them a bike. Everyone loves to cycle, and with a sporty adaptive tricycle from Rifton, everyone can have a set of wheels giving them the freedom to take off and go.

There's hardly a more fun, motivating activity for improving leg strength, balance, fitness and endurance. And in addition to the great physical benefits, trike riding encourages socialization and the development of friendships. Clients with special needs can zoom around right alongside their peers on a tricycle that's as cool and colorful as anything on the sidewalk.

And of course, our adaptive tricycles are designed to be flexible and adjustable for all ages with no tools required.



“ My student received the trunk support and rear steering bar for his tricycle. He is cognitively impaired and blind, and now he can ride just like other children. The rear steer is fantastic and allows him to ride faster. Now his tricycle allows him to be a part of society and join his peers. ”

Rachel Jodway
Dearborn Public School District, MI

Tricycle (R130)
with trunk support system
and rear steering bar



TRICYCLES



With the rear steering bar the caregiver guides from behind, enabling the rider to see ahead.



Transfers are made easy with the low step, flip-away loop handlebar and removable laterals.



The front guide bar enables interaction between rider and companion.



A great way to exercise!

The stationary stand elevates the back wheels of the tricycle enabling Roger to pedal in place, and exercise his lower extremities. As his muscles move through the cycling motions, they are flexed, extended and stretched. This range of motion is essential therapy for people with disabilities. Additionally, Roger can practice head and trunk control while he pedals in place.

Accessories



Seats: Choose between two sizes, small or large. Both sizes are upholstered for comfort.

Small: 10" wide x 10" long

Large: 16" wide x 12" long



Handlebars: Choose a traditional handlebar with conventional handgrips or a padded loop handlebar for extra support.



Trunk support system stabilizes a rider with poor trunk control. The back support assembly adjusts for height and seat depth. Laterals adjust for trunk width, and can be removed for easy transfer. A butterfly harness provides anterior support.

One size only: 14" wide x 15½" high



Additional backrest with security strap provides extra support.



Headrests: Choose between winged, contoured or flat headrests. They adjust horizontally and vertically for optimum positioning and attach to the trunk support system, tool-free.



Handbrake comes standard on the large trike and is an option for the medium trike. It attaches anywhere on the padded loop handlebar. The handbrake must be factory installed.





Communication tray is easy to clean and designed to hold many things, including a communication device.

Small: 8½" x 12½"

Large: 11½" x 13½"

Max. working load (lbs.) Small: 10 Large: 15



Abductor provides comfortable abduction if needed. Just set it in place, tool-free. The two larger sizes are height adjustable with a snap button.



Front guide bar attaches securely to the handlebar and lets a caregiver guide the tricycle from the front. It enables eye contact between the caregiver and rider. Ideal for beginners who can't steer themselves.



Rear steering bar lets a caregiver guide the tricycle from behind, providing the rider with a feeling of greater independence. Ideal for beginners who can't steer themselves.



Front pulley maintains a level pedal position for riders whose extreme tone forces the front of the pedal downward.



Stationary stand converts any Rifton tricycle into a stationary trainer for pedaling practice or rainy-day activity.

Raises rear axle: 6¾", 7¾" or 9"



Features of the Tricycle

Choice of handlebar.

The foam-padded loop handlebar is adjustable in height, and positions at any angle. Conventional handlebars adjust in height, and have standard handgrips.

Easily engaged parking

brake grips both back wheels and keeps the trike firmly in place.

Sturdy wheels have puncture proof BMX-type tires. The direct drive and low gear ratio make forward and backwards pedaling easier and safer.



Comfortable and safe.

Both small and large seats are made with comfort and safety in mind.

Easy access.

Open access combined with a low transfer step make it easy to get on and off the trike.

A place for your gear.

There's a tote box at the back of your trike for storage or for transporting items.

Self-leveling pedals

are forward of the seat for comfortable reach and efficient pedaling. Feet are secure on the pedals with hook and loop straps.



All three sizes are available in four colors: lime, red, blue and raspberry.