

**LECKEY<sup>o</sup>**

# Corner Sitter

USER MANUAL





The Leckey Corner Sitter has been designed to improve posture and offer a better quality of life. This manual shows you how you can quickly and easily make use of all of the functions. The instructions on the safety and maintenance of the product will ensure that you will enjoy the use of this product for a long time.

## Contents

1. Intended Use
2. Declaration of conformity
3. Terms of warranty
4. Product history record
5. Product training record
6. Safety information
7. How to unpack and assemble
8. Clinical setup for postural management & daily use
9. Cleaning & care information
10. Daily product inspection
11. Annual product inspection
12. Reissuing Leckey products
13. Product servicing
14. Technical information

## **1 Intended Use**

The Leckey Corner sitter is intended for children with disabilities from the age of 11/2 -14. It can be used for long sitting which aids the therapeutic stretching of the ham strings. The Corner Sitter also provides minimal support for the pelvis and lower spine to promote sitting balance and function for the trunk and extremities from a sitting position.

## **2 Declaration of Conformity**

James Leckey Design Ltd. as manufacturer with sole responsibility declares that the Corner Sitter conforms to the requirements of the 93/42/EEC Guidelines and EN12182 Technical aids for disabled persons. General requirements and test methods.

## **3 Terms of Warranty**

The warranty applies only when the product is used according to the specified conditions and for the intended purposes, following all manufacturers' recommendations (also see general terms of sales, delivery and payment). A two year warranty is provided on all Leckey manufactured products and components.

## **4 Product History Record**

Your Leckey product is classified as a Class 1 Medical device and as such should only be prescribed, set up or reissued for use by a technically competent person who has been trained in the use of this product. Leckey recommend that a written record is maintained to provide details of all setups, reissue inspections and annual inspections of this product.

## **5 Product Training Record (Parents, Teachers & Carers)**

Your Leckey product is a prescribed Class 1 Medical Device and as such Leckey recommend that parents, teachers and carers using the equipment should be made aware of the following sections of this user manual by a technically competent person:

### **Section 6**

Safety Information

### **Section 8**

Clinical setup for postural management & daily use

### **Section 10**

Daily Product Inspection

Leckey recommend that a written record is maintained of all those who have trained in the correct use of this product.

## 6 Safety Instructions



1. Always read instructions fully before use.



2. Users should not be left unattended at any time whilst using Leckey equipment.



3. Only use Leckey approved components with your product. Never modify the product in any way. Failure to follow instructions may put the user or carer at risk and will invalidate the warranty on the product.



4. When placing your client in the Corner Sitter, both for positional and safety reasons, always secure the hip belt first.



5. If in any doubt to the continued safe use of your product or if any parts should fail, please cease using the product and contact our Customer Care Team or local dealer as soon as possible.



6. Never leave the product on a sloping surface, greater than 5 degrees. Always remember to lock all the castors.



7. The product contains components which could present a choking hazard to small children. Always check that locking knobs and bolts within the child's reach are tightened and secure at all times.



8. Leckey products comply with fire safety regulations in accordance with EN12182. However the product contains plastic components and therefore should be kept away from all direct sources of heat including naked flames, cigarettes, electric and gas heaters.



9. Do not place hot objects on the tray, not greater than 40 degrees Celsius



10. Leckey products comply with fire safety regulations in accordance with EN12182. However the product contains plastic components and therefore should be kept away from all direct sources of heat including naked flames, cigarettes, electric and gas heaters.



11. Clean the product regularly. Do not use abrasive cleaners. Carry out maintenance checks on a regular basis to ensure your product is in good working condition.



12. The product is designed for indoor use and when not in use should be stored in a dry place that is not subjected to extremes of temperature. The safe operating temperature range of the product is +5 to +40 deg Celsius.


# How to unpack and assemble

Congratulations on purchasing your Corner Sitter. Your Corner Sitter will be boxed and wrapped in bubble wrap packaging on receipt. The Corner Sitter will be supplied with a hip belt and pommel and with your choice of accessories.



The Corner Sitter will arrive folded.  
To assemble the Corner Sitter lift up the  
backrest to the required position and secure  
all 4 knobs.

The Corner Sitter is a portable floor seating  
unit with a handle at the back to make  
carrying easier.

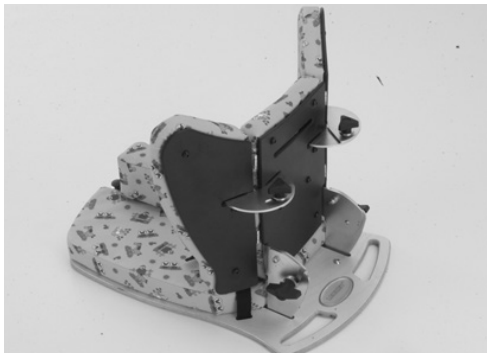

# Clinical setup for postural management & daily use

The clinical setup of the product should be completed by a technically and clinically competent person who has been trained in the use of the product. Leckey recommend a written record is maintained of all clinical setups for this product.

Parents and carers should be shown how to make frequent adjustments and made aware of the safety checks in Section 6 by a technically and clinically competent person who has been trained in the use of the product. Leckey recommend that a written record is maintained of all parent and carers who have been trained in the use of this product.





## Lateral Adjustments

The sides of the Corner Sitter can be adjusted in towards the user to give additional lateral support. Simply release the two back adjustment knobs (a) (one on either side), select the position you require and re-tighten.



## Pommel

To adjust the depth of the pommel, pull up the pommel adjustment pin (a), select the position you require and release the pin.



## Back Angle Adjustment

A degree of back recline and prone angle can be achieved by loosening the adjustments knob (a) (one at either side), selecting the position you require and re-tightening the knobs. Please note that the backrest should be supported by the carer when adjusting the angle while the user is seated in the chair.

## Tray height and Angle Adjustment



The Corner Sitter Table is height and angle adjustable. To adjust the angle of the table loosen the two table top adjustment knobs (a) (one on each side), select the position you require and re-tighten the knobs securely.

To adjust the height of the table, press in poppers and position the leg to the relevant slot to obtain required height.



## Multi Adjustable Contoured High Back

If a high back has been specified at the time of order, a bracket to fit the high back will already be attached. If the high back has been ordered separately then the high back bracket should be attached to the back of the seat by securing it with two allen bolts supplied. The high back is height adjustable by loosening the knob (a). Once the required position is achieved knob (a) should be re-tightened securely. To adjust the depth loosen the allen bolt (b), position as required and then re-tighten securely.

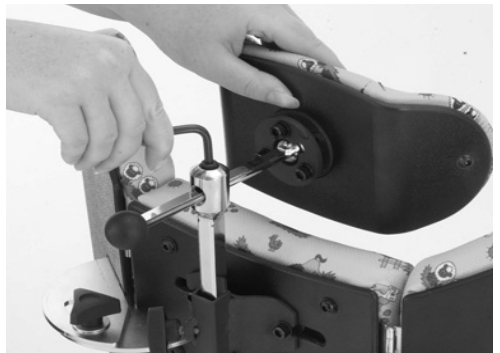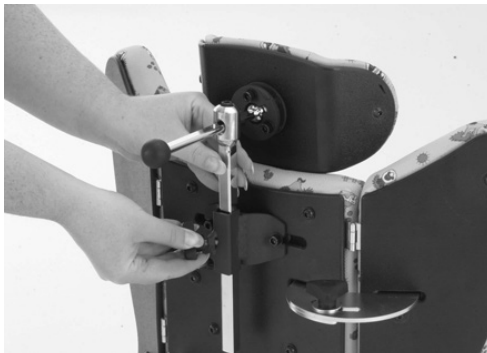

## 9 Cleaning & Care Information

### How to Maintain

When cleaning the product we recommend that you use only warm water and a non-abrasive detergent. Never use organic solvents or dry cleaning fluids.

### Upholstery and fabrics

1. The upholstery and fabrics can be cleaned by hand whilst in place. When cleaning we recommend that you use only warm water and a nonabrasive detergent.

2. The best cleaning method is a 'wipe & dry' technique.

3. Staining should be removed as quickly as possible with absorbent cloth, towels or a sponge. Routine soap and warm water sponging is effective for ordinary soiling and minor spills. Be careful not to over wet the fabric, as this will cause the staining to spread.

4. Antiseptic cleaning agents can be used on more stubborn stains. These may require a safe solvent such as Isopropyl Alcohol or Mineral Spirit. A half cup of household bleach to 5 litres of water can also be used as a useful disinfectant.

5. Always ensure the product is dry before use.

### Metal and plastic and wooden components

1. Soap and water or antibacterial spray can be used for daily cleaning.

2. For deep cleaning a low pressure steam cleaner can be used.

3. Do not use solvents to clean plastic wooden or metal components.

4. Make sure the product is dry before use.

5. Check all wooden components regularly to ensure there are no cracks or splitters.

## 10 Daily Product Inspection

### (Therapists, Parents & Carers)

We recommend that daily visual checks of the equipment are carried out by therapists, carers or parents to ensure the product is safe for use. The recommend daily checks are detailed below.

1. Ensure all adjustment knobs and bolts are in place and secure.
2. Check all upholstery for signs of wear and tear.



**If in any doubt to the continued safe use of your Leckey product or if any parts should fail, please cease using the product and contact our Customer Care Team or your local dealer as soon as possible.**

## 11 Annual Product Inspection

### (Therapist, Technician, VIDA Product Specialist, Dealer)

Leckey recommend that each product should be subject to a detailed inspection at least once a year and every time the product is reissued for use. This inspection should be carried out by a technically competent person who has been trained in the use of the product and should include the following checks as a minimum requirement.

1. Visually check the structure of the product paying attention to weld points on the frame ensuring there are no signs of fatigue or cracking around the welds.

2. Leckey recommend that a written record is maintained of all annual product inspections.



**If in any doubt to the continued safe use of your Leckey product or if any parts should fail, please cease using the product and contact our Customer Care Team or your local dealer as soon as possible.**

## 12 Re-issuing Leckey Products

Most Leckey products are assessed and ordered to meet the needs of an individual user. Before reissuing a product we recommend that the therapist prescribing the product has carried out an equipment compatibility check for the new user and has ensured that the product being re-issued contains no modifications or special attachments.

A detailed technical inspection should be carried on the product prior to re-issuing. This should be carried out by a technically competent person who has been trained in the use and inspection of the product. Please refer to section 11 for the required checks to be carried out.

Ensure the product has been cleaned thoroughly in accordance with section 9 of this manual.

Ensure a copy of the user manual is supplied with the product. A copy can be downloaded from our website [www.leckey.com](http://www.leckey.com)

Leckey recommend that a written record is maintained of all product inspections carried out during the reissue of the product.



**If in any doubt to the continued safe use of your Leckey product or if any parts should fail, please cease using the product and contact our Customer Care Team or your local dealer as soon as possible.**

## 13 Product Servicing

Servicing of all Leckey products should only be carried out by technically competent persons who have been trained in the use of the product.

All international service enquiries should be directed to the appropriate Leckey distributor who will be delighted to assist you. For further information on Leckey distributors please visit our website [www.leckey.com](http://www.leckey.com).

# 14 Technical Information

## Product & Accessory Codes

The corner sitter comes as standard with a hip belt and pommel. The size 1 comes in Meadow Fabric and the size 2 and 3 in Mondeo Blue.

LCS/1 - Corner Sitter size 1

LCS/2 - Corner Sitter size 2

LCS/3 - Corner Sitter size 3

LCT/# - Corner Sitter Table

LCT/#/104-\* Multi Adjustable contoured high back

# Please select size at time of order

\* Please select colour at time of order

# 15 Technical Data

	Size 1	Size 2	Size 3
Age (approx)			
min	1.5	5	8
max	6	9	14

	Size 1		Size 2		Size 3	
	mm	inches	mm	inches	mm	inches
Overall User Height	1165	45.9	1330	52.4	1630	64.2
Seat Base						
length	335	14	460	18.1	550	21.6
width	380	15	425	16.7	475	18.7
Height in Folded Position	180	7	180	7	180	7
Weight of Product	9kg		11kg		13kg	
Backrest Height	300	11.8	400	15.7	500	19.7
High Back Height	120	4.7	140	5.5	160	6.3
High Back Adjustment	170	4	170	5.3	170	6.7
Pommel Adjustment	50	2.7	75	2.7	75	2.7
Max User Weight	30kg		40kg		50kg	
Degree of Angular Adjustment on Backrest						
prone	15		15		15	
recline	20		20		20	
Tray Sizes	640	25	640	25	640	25
	x450	x17.7	x450	x17.7	x450	x17.7
Tray Height (from ground)						
min	220	8.8	220	8.8	220	8.8
max	320	12.8	320	12.8	320	12.8





Distributed by



**VIDA Global**

19C Ballinderry Road,  
Lisburn, BT28 2SA  
Northern Ireland,  
United Kingdom

**T:** 028 9260 0750

**F:** 028 9260 0799

**E:** [info@vidaglobal.co.uk](mailto:info@vidaglobal.co.uk)

**W:** [vidaglobal.co.uk](http://vidaglobal.co.uk)



24 hour postural  
care for babies,  
kids & adults.  
Sleeping, Sitting,  
Standing, Walking,  
Moving, Bathing,  
Toileting.

A member of the **LECKEY** Group



LS565-04