

The springy walking support MEYWALK® 2000



Quickly and stepless adjustment of the height of the trunk support and the seat.



Out of the wheelchair...

MEYWALK® 2000
Medium



MEYWALK® 2000
Large



...and away !

MEYWALK® 2000 from Meyland-Smith in Denmark is the improved version of the famous MEYWALK® walking support. The height of the trunk support and seat are now adjusted quickly and stepless with spanner grips. MEYWALK® 2000 Medium is for children from 7 to 14 years of age, and MEYWALK® 2000 Large is for young people from 14 years of age and adults. The seat and trunk support has spring suspension to encourage a better and more natural gait. The seat unit can be lowered by using a swing bar, making it easier for the user to get in and out of the walking support and putting less strain on the carers. The body posture of the user can be set in any position from vertical to bent forwards by 15°, making it possible to find the body inclination that gives the user the best control. MEYWALK® 2000 is made with materials of high quality, that guarantee long life, maximum reliability and a minimum of maintenance. MEYWALK® 2000 is therefore suitable for indoor as well as outdoor use.

MEYWALK® 2000 comes complete with seat, trunk support, rear stop, handle bar, fender wheels and parking brakes.

MEYWALK® 2000 has a build in raising/lowering system, making it easier for the user to get in and out of the walking support. Simply release the locking buttons at the front of MEYWALK® 2000 and swing up the swing bar. This lowers the seat unit 15 cm (6") on MEYWALK® 2000 Medium and 25 cm (10") on MEYWALK® 2000 Large. When the user is in position, the seat unit is raised again by pressing the swing bar down until it locks with a audible click. Due to the integral gearing in the raising/lowering system, it only needs one third of the user's weight to push down the swing bar.

The seat and trunk support on MEYWALK® 2000 have spring suspension, and the springs are obtainable in four different strengths, which should be **selected according to the weight of the user:**

- Min. 20 kg (45 lbs.) = White springs
- Min. 45 kg (100 lbs.) = Yellow springs
- Min. 65 kg (145 lbs.) = Blue springs
- Min. 80 kg (175 lbs.) = Red springs

Please inform spring strength when ordering!

The trunk support on MEYWALK® 2000 is obtainable in five different sizes, which should be **selected according to the measure around the thorax of the user:**

- Max. 70 cm (27") in circumference
- Max. 90 cm (35") in circumference
- Max. 105 cm (41") in circumference
- Max. 120 cm (47") in circumference
- Max. 130 cm (51") in circumference

Please inform trunk support size when ordering!

Construction

- Frame manufactured from steel tubes, powder coated.
- Components and accessories black chromium plated / electro-galvanised.
- Solid (purbane) wheels.
- Rims and front wheel forks in black hard plastic with ball bearings.
- Paddings of black water-resistant foam rubber coating.

(We reserve the right to make changes in design and construction.)

Technical data

MEYWALK® 2000	Medium:	Large:
Length:	95 cm (37,5")	95 cm (37,5")
Width:	70 cm (27,5")	70 cm (27,5")
Seat/groin height:	58 - 83 cm (23" - 32,5")	72 - 103 cm (28" - 40,5")
Trunk support height:	91 - 115 cm (36" - 45")	105 - 136 cm (41" - 53,5")
Wheel size, front:	200 x 50 mm (8" x 2")	200 x 50 mm (8" x 2")
Wheel size, rear:	400 x 50 mm (16" x 2")	400 x 50 mm (16" x 2")
Weight:	25 kg (55 lbs.)	26 kg (57 lbs.)
Max. load:	100 kg (220 lbs.)	100 kg (220 lbs.)

Accessories

- Anti-tip supports
- Extra long spacer tubes (Gives 6 cm (2") extra height - **only for MEYWALK® 2000 Large**)
- Height reducing fittings (Gives 8 cm (3") less height)
- Drag brakes
- Non-reverse brakes (can be deactivated)
- Leg guides
- Swivel locks for front casters (can be deactivated)
- Hip pads
- Crossbar - no seat option
- Leg separation plate



MEYLAND-SMITH A/S

INDUSTRIVEJ 27
DK-9830 TAARS

Tel: +45 98 96 19 85 • Fax: +45 98 96 19 86
www.meyland-smith.com • info@meyland-smith.com



The locking buttons are released...



...and the seat unit is lowered !

Frame colours



Turquoise (RAL 5021)



Navy blue (RAL 5022)

Padding colour




Black



MEYWALK® 2000 Medium.



Please get in !
Now with a new more simple rear stop.

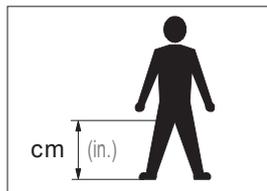

This is how the seat/groin height is measured.