

Activity Chair

In a few short years the Activity Chair has become the classroom standard for children and young adults with special positioning needs. Not only that, but parents who see what this chair can do in school are getting them for use at home. We've also seen a growing number of speech and feeding specialists adopt this chair in their hospital-based clinics. It's safe to say the Activity Chair has truly revolutionized adaptive seating and continues to delight therapists with its versatility for people of all shapes and sizes, all conditions and special needs.

i

Find inservice videos, letters of medical necessity and informative articles at: www.rifton.com/activitychair

Your Activity Chair provides incredible ease-of-use with all the reliability and durability that I've come to trust from Rifton. It gives children the ability to participate in age-appropriate sitting activities which increases self-confidence. All the secondary positioning components meet the demands of even the most involved diagnoses. It's a great chair.

Jeremy Elliott, ATP/CRTS Cumberland, Maryland





The mini kit extends the size range of the small Activity Chair. After this child has outgrown the mini kit, he will continue to use the chair for many more years.

Activity chair features



Tilt-in-space



Adjustable back



Standard base



Spring option The springs on the standard base (in both backrest and seat) allow for self-generated, dynamic motion – ideal for clients with autism. The spring column can be locked when not in use.



Caster swivel lock The swivel lock prevents the chair from drifting sideways when it is pushed for longer distances.

Continue of the second seco





Tilt-in-space



Adjustable back

Hi/lo base



Adjust while in use The Activity Chair can be easily adjusted without tools, and while a client is in the chair. This saves time and makes adjustments much simpler to gauge.



Easy height adjustment The large and medium hi/lo bases are height adjustable with the foot pump.

A chair for everything

The Activity Chair's tilt-in-space feature allows easy transitions between forward leaning for task participation and unimpeded interaction, and reclining for rest or a higher level of support.







All Rifton Activity Chairs have tilt-in-space.

Whether you choose the standard or hi/lo base, you'll get Rifton's quick-adjusting, tilt-in-space feature. The standard base tilts 15° forward and 15° back, while the hi/lo tilts 15° forward and 25° back. Tilt-in-space is a valuable feature because it encourages sit-to-stand and offers a variety of resting positions. Best of all, our tilt-in-space can be activated while your client is in the chair.

@rifton



This large standard chair is in anterior tilt so that Emily can participate in sewing. She's using the slim cut butterfly harness and the pelvic harness.



The tilt-in-space feature accomodates users who need a higher degree of head support.

Infinitely adjustable for every need

The favorite of feeding clinics

Rifton's Activity Chair can be either a simple, basic chair for a client who needs just a little support, or it can offer significant support for more involved individuals. The height options allow the client to be fed in a quiet, individual setting or at the family dinner table. Because you can measure the angle of the tilt-in-space and backrest recline, home caregivers can exactly duplicate the angles recommended by therapists.

Each size of Activity Chair has so much growth potential that it will provide many years of service. And it is easy to clean – essential for a feeding chair. The tray clips on and off with one hand.

Quickly adjustable tilt-in-space during feeding is beneficial. Therapists may start out the session doing oral motor exercise with the client upright and then tilt back to reduce the need for trunk control. They can then adjust the headrest forward to keep the head in a neutral position.

All these features make it the ideal feeding chair.





From circle time at floor level to the cafeteria table, the hi/lo base chair makes it easy for these children to get on the same level with their peers.





Perfect for the MOVE program



When used with forearm prompts, the Activity Chair becomes a forward-leaning instructional seating device as used in the MOVE program.

Transfers are easy with the Activity Chair



Easy lateral transfers: Remove the armrests and raise the hi/lo chair to the ideal transfer height for secure, safe lateral moves.



Sit-to-stand transfers: Push the footboard back out of the way and tip the tilt-in-space forward for easy sit-to-stand transfers.



Patient-assist transfers: Lower the footboard of the hi/lo chair until it is flat on the floor. Because the footboard is strong enough for stand and pivot transfers, clients can assist in their own transfers.

Components



Choice of arm supports

Armrests provide lateral boundaries as well as upper extremity weight-bearing assist to aid trunk control. Rifton's armrests can be raised or lowered individually and can be set at angles from horizontal to 30° up or down. The tray attaches to the armrests.

Small 2¼"x 8¾", large 3"x 12"

Forearm prompts facilitate head and trunk control while sitting and secure distal placement of upper extremities. This enables the use of the pelvic girdle and trunk core muscles for improved postural stabilization and strengthening. Rifton's forearm prompts are fully adjustable for optimal positioning.

Padded area: small 4"x 7", large 4½"x 8" Range of adjustment from elbow to fist: small 8"-12½", large 10½"-16"







Choice of adjustable legs

Adjustable legs (long and short): Height-adjustable in 1" increments, these legs allow you to position your client at optimal height. The grippy tips of the stationary legs keep your client from sliding and moving across the floor, even with dynamic rocking.

Adjustable legs with casters (long and short): Heightadjustable in 1" increments, these legs allow you to position your client at optimal height. The casters make your standard base chair easy to move when transitioning from one classroom to another.





Pads: Comfortable seat and backrest pads are essential for providing cushioned support and preventing decubiti. The backrest and seat pads are made of polyurethane foam covered with a cleanable, healthcare-grade polyurethane-coated fabric. The seat pad can be turned front to back for longer wear. Pads are available in 6 colors.







Headrests optimally position clients with poor head control. Choose a flat, contoured or winged headrest. The winged headrest adjusts from almost flat to 90°. All headrests adjust front to back and up and down using Rifton's unique two-knob adjustment system.

Flat and contoured: 9"x 5" Winged: 16"x 5" when flat to 6" between wings at 90°





Laterals: Necessary for clients who need good upper body support, laterals are important for proper alignment of the spine. Laterals have independent vertical, lateral and angle adjustment.

Small 5"x 4", large 6½"x 5"





Laterals with chest strap: With the addition of a wide chest strap to the laterals, clients with limited upper torso control gain added trunk support and security.





Push handles: Recommended for all chairs with wheels, push handles make it easy for a caregiver to maneuver and transport a client in the Activity Chair from one place to another.





Tray: The tray attaches to the armrests and provides a work, play and feeding surface with raised edges. It can be set at angles ranging from horizontal to 30° up or down, and can be raised or lowered to meet your client's needs.

Small 19" wide x 18" deep, **med.** 22" wide x 20" deep **Large** 25½" wide x 22½" deep



Handhold: The handhold attaches to the tray for arm and hand positioning to control spasticity. Handholds are especially useful for clients with spastic muscle tone and are helpful for proper joint alignment.





Components (continued)



Chest strap: The chest strap provides added trunk support with its wide and comfortably padded strap. The chest strap is for clients with limited upper torso control who need extra support and security.

Max. circumference: small 34", med. 40", large 52" Width of strap: small 3", med. 3¾", large 4"





Butterfly harness: When adjusted to fit snugly and comfortably, the butterfly harness provides maximum anterior support giving security, safety and freedom of movement.

Slim-cut butterfly harness performs the same positioning function as the regular butterfly harness. The slim cut butterfly harness is an alternative especially designed for larger female clients.

Thigh belt: The thigh belt provides knee adduction and additional support and security for the client's thighs.

Max. circumference: small 32", med. 41", large 52"









Pelvic harness: An alternative to a typical seat belt, Rifton's unique pelvic harness firmly positions a client's pelvis by securing hips and upper thighs without pressure on the abdomen. The pelvic harness provides a stable, comfortable base for postural control and encourages proper alignment of the spine. It also controls extreme extensor spasticity that can result in sacral sitting.

Max. thigh circumference: small 22", med. 25", large 31"





Hip guides attach to the arm supports. They provide a snug, comfortable fit giving the client security and safety. Placed at the sides of the pelvis to align a client's torso, they provide maximum support of trunk control while sitting. Hip guides can also be important for proper alignment of the spine. They independently adjust vertically and horizontally. For transfers, hip guides remove with the arm supports. **Dimensions:** small 3¼"x 5½", large 4"x 9¼"



erifton



Abductor: The abductor separates knees and helps to control spastic muscles that cause legs to scissor. The abductor can be adjusted forward and back in three positions to improve postural control.

Dimensions: small 3"x 6"x 3½", large 4"x 7½"x 4"

Adductors: Adductors provide a comfortable lateral boundary for client's knees and maintain proper body and joint alignment by supporting each leg individually.

One size only: 5" high x 6" long







Leg prompts are helpful for clients with limited muscle control and maintain body and joint alignment when sitting by supporting each leg individually with a strap that passes around the front. Leg prompts move in and out and laterally for maximum adjustability.

Max. circumference: small 14", med. 16", large 18"





Footboard: The footboard facilitates weight-bearing through the legs and feet during functional sitting activities, and can be used with sandals and ankle straps to meet specific positioning needs. The footboard swings back out of the way for sit-to-stand transfers or can be used as a step for transfers.



Footboard lift provides additional elevation, enabling shorter legs to reach the footboard. Available for all three chair sizes.

Small adds 3", med. and large add 4" $\,$



Ankle straps (require footboard) are helpful for clients with limited muscle control and excursion movement. They allow a moderate range of movement while improving independence in postural control.

Note: Ankle straps cannot be used together with sandals. **Max. circumference:** small 12", large 16¹/₂"



Components (continued)



Sandals (require footboard): For clients who are unable to control leg movement, sandals adjust to position client's feet correctly for optimal body alignment.

Small 3"x 7", med. 4"x 9", large 5"x 10½"





Wedges (require sandals) can accommodate clients with specific ankle positioning needs or uneven leg lengths. Use them to raise heels or toes, or combine two wedges under one sandal to add flat lift.

Lumbar and seat support kit: This kit lets you adapt the contours of the seat and back for a specific client's needs. They can be cut to length and attached where needed under the seat and back

pads with adhesive-backed hook and loop strips.

Thickness: small 1", med. 1¼", large 1½"








Backrest filler pad fills the space between the bottom edge of the backrest and the top edge of the seat when backrest is in one of the top three positions. It provides additional lower back support.





Mini kit: Consisting of a backrest insert pad, footboard lift and a mini trunk support, the mini kit supports children as short as 28" tall. You can start positioning children correctly at an earlier age, and continue to use the small chair for years before it's outgrown.

Dimensions: Footboard lift adds 3", trunk support max. circumference is 26", backrest pad is 1" thick

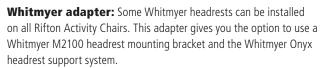

Note: Rifton supplies only the adapter and T20 power tip. Whitmyer mounting brackets and headrests must be purchased from a Whitmyer supplier.







Choose the chair that's right for you.

Standard base



R820 Small



R840 Medium



R860 Large

Hi/lo base



R830 Small

R850 Medium

R870 Large

Room to grow

Small chair with mini kit at its smallest dimensions

Backrest height 12½"

- 2 Trunk support width 5½"
- Seat depth 7"
- 4 Seat width 7"
- G Armrest height above seat 5"
- G Seat height above footboard 6"



Small chair at its largest dimensions

- Backrest height 15½"
- Distance between laterals 11½"
- Seat depth 12"
- 4 Seat width 12"
- G Armrest height above seat 7½"
- G Seat height above footboard 12"



32-48

28-38

R830 small

Hi/lo base



Small standard base

R820 Rifton Activity Chair

User dimensions (inches)

Height with mini kit



Key user dimension: height

The user's overall height is a general guide to help you select the appropriate chair. Choose the model that allows for growth.

Important: Make sure that seat width, depth, and height are adequate for each individual user, and that the user's weight does not exceed the maximum weight recommended.

Item dimensions (inches)	R820 small Standard base
Frame width	short legs: 21
	long legs: 23
	short legs w/ casters: 221/2
	long legs w/ casters: 231/2
Seat height above floor	short legs: 91/2-121/2
	long legs: 181/2-211/2
	short legs w/ casters: 131/2-161/2
	long legs w/ casters: 181/2-211/2
Seat angle (tilt-in-space)-degrees	15° forward, 15° back
Backrest angle-degrees	5° forward, 20° back
Seat height above footboard with footboard lift	9–12 6–9
Seat width with hip guides without hip guides	7-9 12
Seat depth with mini kit	8-12 7-11
Armrest height above seat	5-71/2
Trunk support width	51/2-111/2
Backrest height	121/2-151/2
Headrest height above seat	141/2-21
Max. working load (lbs)	75



32-48

28-38

Small hi/lo base

R830 Rifton Activity Chair

User dimensions (inches)

Height with mini kit	
	Key user dimension: height The user's overall height is a general guide to help y

you select the appropriate chair. Choose the model that allows for growth. Important: Make sure that seat width, depth, and height are

adequate for each individual user, and that the user's weight does not exceed the maximum weight recommended.

Item dimensions (inches)

Frame width	26
Seat height above floor	10-25
Seat angle (tilt-in-space)-degrees	15° forward, 25° back
Backrest angle-degrees	5° forward, 20° back
Seat height above footboard with footboard lift	9-12 6-9
Seat width with hip guides without hip guides	7-9 12
Seat depth with mini kit	8-12 7-11
Armrest height above seat	5-7½
Trunk support width	51/2-111/2
Backrest height	121/2-151/2
Headrest height above seat	14½-21
Max. working load (lbs)	75



11

2019 order form for small

Use dimension chart to select appropriate size.

Jse dimension chart to select approp	ria	le size.		F		
			Standard R820 small	Ala	Hi/lo R830 small	200
* Seat and back includes seatbelt	유	Adjustable	R821	\$540 🗌	R831	\$1,100
(Hi/lo includes footboard and push handles)	CHOOSE	Adjustable w/ back spring	R822	\$615 🗌	R832	\$1,200 🗌
*Base	£	Without spring	R823	\$315 🗌	R833	\$1,880
	CHOOSE	With spring	R824	\$370 🗌	N/A	
*Arm supports (pair)		Armrests	R827	\$220 🗌	R827	\$220 🗌
	CHOOSE	Forearm prompts	R809	\$540 🗌	R809	\$540 🗌
*Adjustable legs		Short	R803	\$43 🗌	N/A	
	£	Long	R894	\$73 🗌	N/A	•••••
	CHOOSE	Short with casters	R805	\$265 🗌	N/A	
		Long with casters	R896	\$295 🗌	N/A	•••••
*Pads		Tan	R815	\$220 🗌	R815	\$220 🗌
		Blue	R816	\$220 🗌	R816	\$220 🗌
	£	Red	R817	\$220 🗌	R817	\$220 🗌
	CHOOSE	Pink	R971	\$220 🗌	R971	\$220 🗌
		Purple	R972	\$220 🗌	R972	\$220 🗌
		Green	R973	\$220 🗌	R973	\$220 🗌
Headrest	~	Contoured	R897	\$220 🗌	R897	\$220 🗌
	CHOOSE	Flat	R898	\$205 🗌	R898	\$205 🗌
	ň	Adjustable winged	R881	\$270 🗌	R881	\$270 🗌
Laterals (pair)	CHOOSE	Small	R828	\$180 🗌	R828	\$180 🗌
	OSE	Small with strap	R819	\$220 🗌	R819	\$220 🗌
Push handles (recommended for all chairs with wheels)			R878	\$140 🗌	Included	
Tray (requires armrests)		R667	\$220 🗌	R667	\$220 🗌	
Handhold			R649	\$22 🗌	R649	\$22 🗌
Chest strap (wide)			R802	\$75 🗌	R802	\$75 🗌
Butterfly harness			R834	\$105 🗌	R834	\$105 🗌
Thigh belt			R811	\$75 🗌	R811	\$75 🗌
Pelvic harness			R845	\$140 🗌	R845	\$140 🗌
Hip guides (pair)			R814	\$185 🗌	R814	\$185 🗌
Abductor			R884	\$115 🗌	R884	\$115 🗌
Adductors (pair)			R892	\$180 🗌	R892	\$180 🗌
Leg prompt			R854	\$195 🗌	R854	\$195 🗌
Footboard (recommended for chairs with long legs)			R847	\$425 🗌	Included	
Footboard lift			R825	\$210 🗌	R825	\$210 🗌
Mini kit (footboard lift, backrest insert, and trunk support)			R812	\$280 🗌	R812	\$280 🗌
Ankle straps (pair)			R899	\$80 🗌	R899	\$80 🗌
Sandals (pair)			R944	\$80 🗌	R944	\$80 🗌
Wedges (pair, requires sandals)		R697	\$61 🗌	R697	\$61 🗌	
Lumbar and seat support kit		R886	\$61 🗌	R886	\$61 🗌	
Backrest filler pad		R849	\$115 🗌	R849	\$115 🗌	
Whitmyer headrest adapter		R804	\$50 🗌	R804	\$50 🗌	
Total		Total		\$	R830	\$

* Required

Room to grow

Medium chair at its smallest dimensions

- Backrest height 15½"
- Distance between laterals 6½"
- Seat depth 11"
- 4 Seat width 8½"
- 6 Armrest height above seat 7"
- G Seat height above footboard 12" (with footboard lift 8")



Medium chair at its largest dimensions

- Backrest height 19" Distance between laterals 12"
- Seat depth 16"
- ④ Seat width 14"
- S Armrest height above seat 101/2"
- G Seat height above footboard 161/2"





User dimensions (inches)

Medium standard base R840 Rifton Activity Chair



Medium hi/lo base **R850 Rifton Activity Chair**

User dimensions (inches)

Height

Key user dimension: height

The user's overall height is a general guide to help you select the appropriate chair. Choose the model that allows for growth.

Important: Make sure that seat width, depth and height are adequate for each individual user, and that the user's weight does not exceed the maximum weight recommended.

Item dimensions (inches)	R840 medium Standard base
Frame width	short legs: 231/4
	long legs: 241/2
	short legs w/ casters: 241/2
	long legs w/ casters: 251/2
Seat height above floor	short legs: 121/2-161/2
	long legs: 19–23
	short legs w/ casters: 16-20
	long legs w/ casters: 19-23
Seat angle (tilt-in-space)-degrees	15° forward, 15° back
Backrest angle-degrees	5° forward, 20° back
Seat height above footboard with footboard lift	12-16½ 8-12½
Seat width with hip guides without hip guides	8½-11½ 14
Seat depth	11-16
Armrest height above seat	7-10½
Distance between laterals	61/2-12
Backrest height	15½-19
Headrest height above seat	17½-24½
Max. working load (lbs)	150



40-56

Key user dimension: height The user's overall height is a general guide to help you select the

40-56

appropriate chair. Choose the model that allows for growth. **Important:** Make sure that seat width, depth and height are

adequate for each individual user, and that the user's weight does not exceed the maximum weight recommended.

Item dimensions (inches)	R850 medium Hi/lo base
Frame width	27½
Seat height above floor	12-23
Seat angle (tilt-in-space)-degrees	15° forward, 25° back
Backrest angle-degrees	5° forward, 20° back
Seat height above footboard with footboard lift	12-16½ 8-12½
Seat width with hip guides without hip guides	8½-11½ 14
Seat depth	11-16
Armrest height above seat	7-10½
Distance between laterals	61/2-12
Backrest height	15½-19
Headrest height above seat	171/2-241/2
Max. working load (lbs)	150



2019 order form for medium Use dimension chart to select appropriate size. Standard Hi/lo R840 medium R850 medium * Seat and back includes seatbelt R841 Adjustable \$550 R851 \$1,150 CHOOSE (Hi/lo includes footboard and push handles) Adjustable w/ back spring R842 \$630 🗌 R852 \$1,250 🗌 * Base CHOOSE Without spring R843 \$320 🗌 R853 \$2,315 R844 With spring \$385 🗌 N/A * Arm supports (pair) CHOOSE \$225 🗌 R868 R868 \$225 🗌 Armrests Forearm prompts R869 \$560 🗌 R869 \$560 🗌 * Adjustable legs Short R893 \$50 🗌 N/A Long R894 \$73 🗌 N/A CHOOSE Short with casters R895 \$270 🗌 N/A R896 \$295 Long with casters N/A * Pads Tan R835 \$225 🗌 R835 \$225 🗌 Blue R836 R836 \$225 🗌 \$225 🗌 Red R837 \$225 🗌 R837 \$225 🗖 CHOOSE Pink R974 \$225 🗌 R974 \$225 🗌 Purple R975 \$225 🗌 R975 \$225 🗌 R976 \$225 🗌 R976 \$225 🗌 Green Headrest Contoured R897 \$220 🗌 R897 \$220 🗌 CHOOSE Flat R898 \$205 🗖 R898 \$205 🗖 \$270 \$270 🗌 Adjustable winged R881 R881 Laterals (pair) \$180 🗌 \$180 🗌 Small R828 R828 CHOOSE Small with strap R829 \$225 🗌 R829 \$225 🗌 Large R838 \$195 🗌 R838 \$195 🗌 Push handles (recommended for all chairs with wheels) \$140 R878 Included R668 \$250 🗌 R668 \$250 🗌 Tray (requires armrests) Handhold R649 \$22 🗌 R649 \$22 🗌 \$80 🗌 Chest strap (wide) R863 \$80 🗌 R863 CHOOSE \$115 **Butterfly harness** R864 R864 \$115 Regular Slim cut R882 \$115 🗌 R882 \$115 🗌 Thigh belt R865 \$80 🗌 R865 \$80 🗌 **Pelvic harness** R866 \$145 🗌 R866 \$145 🗌 \$190 🗌 Hip guides (pair) R867 R867 \$190 🗌 \$115 🗌 \$115 🗌 Abductor Small R884 R884 CHOOSE Large R891 \$135 🗌 R891 \$135 🗌 Adductors (pair) R892 \$180 🗌 R892 \$180 🗌 Leg prompt R858 \$205 🗌 R858 \$205 🗌 Footboard (recommended for chairs with long legs) R848 \$435 🗌 Included Footboard lift R806 \$215 🗌 R806 \$215 🗌 Ankle straps (pair) R899 \$80 🗌 R899 \$80 🗌 Sandals (pair) \$86 🗌 R954 \$86 🗌 R954 \$61 🗌 R698 \$61 🗌 Wedges (pair, requires sandals) R698 Lumbar and seat support kit R887 \$73 🗌 R887 \$73 🗌 Backrest filler pad R849 \$115 🗖 R849 \$115 🗖 \$50 🗌 \$50 🗌 Whitmyer headrest adapter R804 R804 Total R840 \$ R850 \$

Room to grow

Large chair at its smallest dimensions

- Backrest height 19"
- Distance between laterals 9½"
- Seat depth 15"
- A Seat width 11"
- G Armrest height above seat 7"
- G Seat height above footboard 15" (with footboard lift 11")



Large chair at its largest dimensions

- Backrest height 24"
- Distance between laterals 14"
- Seat depth 20"
- Geat width 17"
- G Armrest height above seat 10½"
- G Seat height above footboard 20"





Large standard base R860 Rifton Activity Chair

Large hi/lo base R870 Rifton Activity Chair

User dimensions (inches)

Height

50-74



Key user dimension: height The user's overall height is a general guide to help you select the

appropriate chair. Choose the model that allows for growth.

50-74

Important: Make sure that seat width, depth and height are adequate for each individual user, and that the user's weight does not exceed the maximum weight recommended.

Item dimensions (inches)	R870 large Hi/lo base
Frame width	29¾
Seat height above floor	16-24
Seat angle (tilt-in-space)-degrees	15° forward, 25° back
Backrest angle-degrees	5° forward, 20° back
Seat height above footboard with footboard lift	15–20 11–16
Seat width with hip guides without hip guides	11–14 17
Seat depth	15-20
Armrest height above seat	7-10½
Distance between laterals	91/2-14
Backrest height	19-24
Headrest height above seat	191/2-291/2
Max. working load (lbs)	250

User dimensions (inches)



Key user dimension: height The user's overall height is a general guide to help you select the

appropriate chair. Choose the model that allows for growth. Important: Make sure that seat width, depth and height are

adequate for each individual user, and that the user's weight does not exceed the maximum weight recommended.

Item dimensions (inches)	R860 large Standard base
Frame width	short legs: 26¾
	long legs: 263/4
	short legs w/ casters: 273/4
	long legs w/ casters: 273/4
Seat height above floor	short legs: 161/2-201/2
	long legs: 20-24
	short legs w/ casters: 161/2-201/2
	long legs w/ casters: 20-24
Seat angle (tilt-in-space) – degrees	15° forward, 15° back
Backrest angle-degrees	5° forward, 20° back
Seat height above footboard with footboard lift	15–20 11–16
Seat width with hip guides without hip guides	11–14 17
Seat depth	15-20
Armrest height above seat	7-101/2
Distance between laterals	91/2-14
Backrest height	19-24
Headrest height above seat	191/2-291/2
Max. working load (lbs)	250

Orifton

Ise dimension chart to select app	propriat	ie size.	Standard R860 large		Hi/lo R870 large	500
* Seat and back includes seatbelt	Ŷ	Adjustable	R861	\$630	R871	\$1,200
(Hi/lo includes footboard and push handles)	CHOOSE	Adjustable w/ back spring	R862	\$730 🗌	R872	\$1,300
* Base		Without spring	R873	\$385	R883	\$2,600
buse	CHOOSE	With spring	R874	\$460 🗌	N/A	\$2,000
* Arm supports (pair)		Armrests	R868	\$225	R868	\$225
	CHOOSE	Forearm prompts	R879	\$580	R879	\$580
* Adjustable legs		Short	R808	\$61 🗌	N/A	
	요	Long	R894	\$73 🗌	N/A	
	CHOOSE	Short with casters	R895	\$270 🗌	N/A	
		Long with casters	R896	\$295 🔲	N/A	
* Pads		Tan	R875	\$235 🗌	R875	\$235 🗖
		Blue	R876	\$235 🗌	R876	\$235 🗖
	윤	Red	R877	\$235 🗌	R877	\$235 🗔
	CHOOSE	Pink	R977	\$235 🗌	R977	\$235 🗌
		Purple	R978	\$235 🗌	R978	\$235 🗌
		Green	R979	\$235 🗌	R979	\$235 🗌
Headrest	0	Contoured	R897	\$220 🗌	R897	\$220
	CHOOSE	Flat	R898	\$205 🗌	R898	\$205 🗌
ŭ m	Ē	Adjustable winged	R881	\$270 🗌	R881	\$270 🗌
Laterals (pair)	0	Small	R828	\$180 🗌	R828	\$180
	CHOOSE	Large	R838	\$195 🗌	R838	\$195 🗌
	ň	Large with strap	R839	\$235 🗌	R839	\$235 🗌
Push handles (recommended for all chair	rs with w	heels)	R878	\$140	Included	
Tray (requires armrests)			R669	\$260 🗌	R669	\$260 🗌
Handhold			R649	\$22 🗌	R649	\$22
Chest strap (wide)			R807	\$85 🗌	R807	\$85
Butterfly harness	CHOOSE	Regular	R889	\$135 🗌	R889	\$135 🗌
	OSE	Slim cut	R826	\$135 🗌	R826	\$135
Thigh belt			R813	\$85 🗌	R813	\$85
Pelvic harness			R846	\$160 🗌	R846	\$160
Hip guides (pair)			R867	\$190 🗌	R867	\$190
Abductor	CHOOSE	Small	R884	\$115 🗌	R884	\$115 🗌
	OSE	Large	R891	\$135 🗌	R891	\$135
Adductors (pair)			R892	\$180 🗌	R892	\$180
Leg prompt			R859	\$215 🗌	R859	\$215
Footboard (recommended for chairs with long legs)			R857	\$445	Included	
Footboard lift			R818	\$220	R818	\$220
Ankle straps (pair)			R855	\$86	R855	\$86
Sandals (pair)			R964 R699	\$98	R964	\$98
	Wedges (pair, requires sandals)			\$67 🗌	R699	\$67
Wedges (pair, requires sandals)						
Wedges (pair, requires sandals) Lumbar and seat support kit			R888	\$80	R888	
Wedges (pair, requires sandals)			R888 R856 R804	\$80 🗆 \$115 🗔 \$50 🗔	R888 R856 R804	\$80\$115 \$115 \$50