

Tricycle

For too long insurers have insisted that biking is purely recreational rather than therapeutic. But you know better, and it's time to insist that your clients depend on the medical benefits that are associated with biking.

We're talking improved leg strength, ROM, balance, fitness, posture and more. And all these benefits are accompanied by the glorious thrill of the ride on a tricycle that's as cool and colorful as anything on the sidewalk.



Find letters of medical necessity and informative articles at: www.rifton.com/trike

student received the trunk support and rear steering bar for his tricycle. He is cognitively impaired and blind, and now he can ride just like other children. The rear steer is fantastic and allows him to ride faster. Now his tricycle allows him to be a part of society and join his peers.

Rachel Jodway Dearborn Public School District, MI

 Tricycle (R140) with simple prompt configuration

Tricycle features



All three sizes are available in four colors: red, blue, pink and lime.

erifton



With the rear steering bar the caregiver guides from behind, enabling the rider to see ahead.



Transfers are made easy with the low step, flip-away loop handlebar and removable laterals.





A great way to exercise!

The stationary stand elevates the back wheels of the tricycle, enabling a client to pedal in place and exercise his lower extremities. As his muscles move through the cycling motions, they are flexed, extended and stretched. Additionally, this client can practice head and trunk control while he pedals in place.



The front guide bar enables interaction between rider and companion.

erifton

Components



Seats

Choose between two sizes. Both are upholstered for comfort and include a seat belt.

Small: 10" wide x 10" long Large: 16" wide x 12" long

Handlebars

Choose a straight handlebar with handgrips or a padded loop handlebar for extra support.









Trunk support system

The trunk support stabilizes a rider with low trunk control. The back rest adjusts for height and seat depth, and laterals can be removed for transfer. A butterfly harness provides anterior support.

Size: 14" wide x 15½" high

Backrest pad

For clients who need less support, this simple backrest pad with a security strap gives freedom of movement. An additional backrest pad can be purchased for lower back support.





Headrests

Choose between winged, contoured or flat headrests. They adjust horizontally and vertically for optimal positioning and attach to the trunk support system, tool-free.





Handbrake

The handbrake is standard on the large trike and an option for the medium trike. It attaches anywhere on the padded loop handlebar.

Note: The handbrake must be chosen at the time of the original purchase. It cannot be added later.





Components (continued)



Communication tray

Abductor

The communication tray is ideal for holding a communication device or other small objects.

Small: 8½" x 12½" Large: 11½" x 13½"

adjustable with a snap button.

Max. working load (lbs.): small 10, large 15


The abductor provides comfortable abduction if needed. The two larger sizes are height









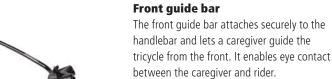







Front pulley

The front pulley maintains a level pedal position for riders whose extreme tone forces the front of the pedal downward.





Stationary stand

The stationary stand converts any Rifton tricycle into a stationary trainer for pedaling practice or rainy-day activity.

Raises rear axle from floor: 6³/8", 7³/8" or 9"



Rear steering bar

The rear steering bar lets a caregiver guide the tricycle from behind, giving the rider a feeling of greater independence.

@rifton

200

2020 order form

Use dimension chart to select appropriate size.

User diı	mensions (inches)	R120 small	R130 medium	R140 large							
Inside leg length		17 – 22	22 – 28	25 – 35							
	Key user dimension: inside leg length The extended leg should reach from seat to pedal comfortably when both feet and torso are strapped into place.										
X	Important: User's weight must not exceed the maximum working load. Rifton tricycles are not intended for clients with strong reflexes or poorly developed trunk balance and head control. Consult your client's therapist before ordering.										
Item dim	ensions (inches)	R120 small	R130 medium	R140 large							
Overall hei	ight (at lowest position)	281⁄2	31	361⁄2							
Overall len	igth	46	52	65							
Overall wid	dth	271/2	271/2	31½							
Wheel dia	meter	13	15	19							
Seat cente	r to extended pedal	17-22	22 – 28	25 – 35							
Seat to top	o of trunk support	151/2-22	151/2-22	15½-22							
Gear ratio	(approximate)	1:1	1:1	1:1							

47

150

8-14

56

8-14

160



				R120 small		R130 medium		R140 large	
* Tricycle frame includes self-leveling pedals with straps, backrest tube with handle, tote box for storage and handbrake for the large only.				\$1,030 🗌	R113	\$1,170 🗌	R114	\$1,980 🗌	
					Red		Red		
Specify color: Red, blue, pink or lime.			Blue		Blue		Blue		
Specify color: Red, blue, pink or lime.					Pink		Pink		
					Lime		Lime		
* Seat (includes seatbelt)	£	Small	R122	\$210 🗌	R132	\$210 🗌	N/A		
	CHOOSE	Large	R128	\$355 🗌	R138	\$360 🗌	(included)		
* Handlebar	CHO	Straight	R126	\$135 🗌	R136	\$145 🗌	R146	\$225 🗌	
	CHOOSE	Loop	R129	\$135 🗌	R139	\$145 🗌	R149	\$225 🗌	
* Back support	CHOOSE	Trunk support system	R147	\$485 🗌	R147	\$485 🗌	R147	\$485 🗌	
	OSE	Backrest pad	R145	\$110 🗌	R145	\$110 🗌	R145	\$110 🗌	
Headrest (requires trunk support system)	_	Winged	R159	\$230 🗌	R159	\$230 🗌	R159	\$230 🗌	
(requires trunk support system)	CHOOSE	Contoured	R156	\$220 🗌	R156	\$220 🗌	R156	\$220 🗌	
	m	Flat	R157	\$210 🗌	R157	\$210 🗌	R157	\$210 🗌	
Handbrakes (Must be factory installed. Cannot be added later.)			N/A		R125	\$110	(included)		
Communication tray		Small	K225	\$190 🗌	K225	\$190 🗌	K225	\$190 🗌	
	CHOOSE	Large	N/A		K245	\$195 🗌	K245	\$195 🗌	
Abductor				\$90 🗌	R131	\$150 🗌	R141	\$170 🗌	
Front guide bar				\$135 🗌	R135	\$135 🗌	R135	\$135 🗌	
Rear steering bar				\$235 🗌	R123	\$235 🗌	R143	\$260 🗌	
Front pulley				\$45 🗌	R144	\$45 🗌	R144	\$45 🗌	
Additional backrest pad				\$110 🗌	R145	\$110	R145	\$110 🗌	
Stationary stand			R155	\$120 🗌	R155	\$120 🗌	R155	\$120 🗌	
Total			R120	\$	R130	\$	R140	\$	

Item weight (lbs)

Distance between trunk laterals

Max. working load (lbs)

* Required